



Nowhere to run, nowhere to hide

Introducing **Advanced Bacteria Detection** for SediVue Dx

A breakthrough innovation for more conclusive and accurate results

Advanced Bacteria Detection with the SediVue Dx® Urine Sediment Analyzer: Two advanced, proprietary technologies that work together to seek out bacteria and help you determine next steps quickly. Both yours at no additional cost.

The SediVue® Bacteria Confirmation Kit
Eliminates suspect presence by chemically dissolving blood cells and crystals that interfere with bacteria identification.


Neural Network 5.0 software release—now powered by 350 million images
With twice as many images as the last release, the SediVue Dx Urine Sediment Analyzer detects bacteria with unprecedented specificity.


.....

Learn how and when to use the SediVue® Bacteria Confirmation Kit today at [idexx.com/svconfirm](https://www.idexx.com/svconfirm)

.....


Advanced Bacteria Detection brings clarity to your results

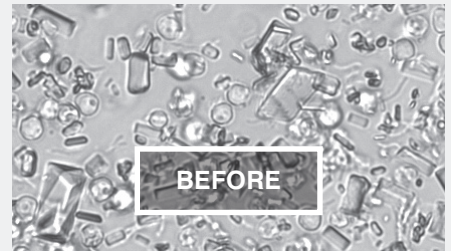

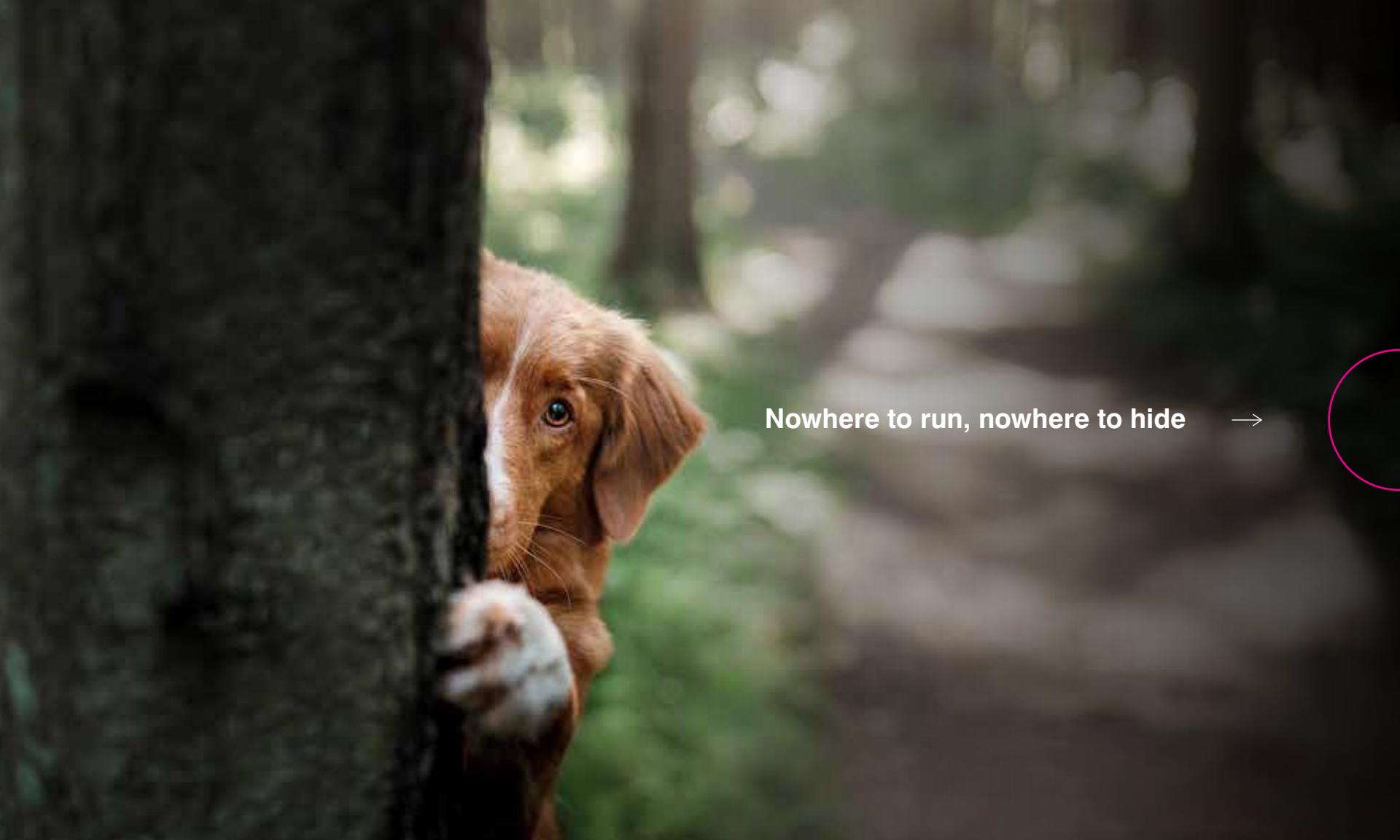

Nowhere to run, nowhere to hide



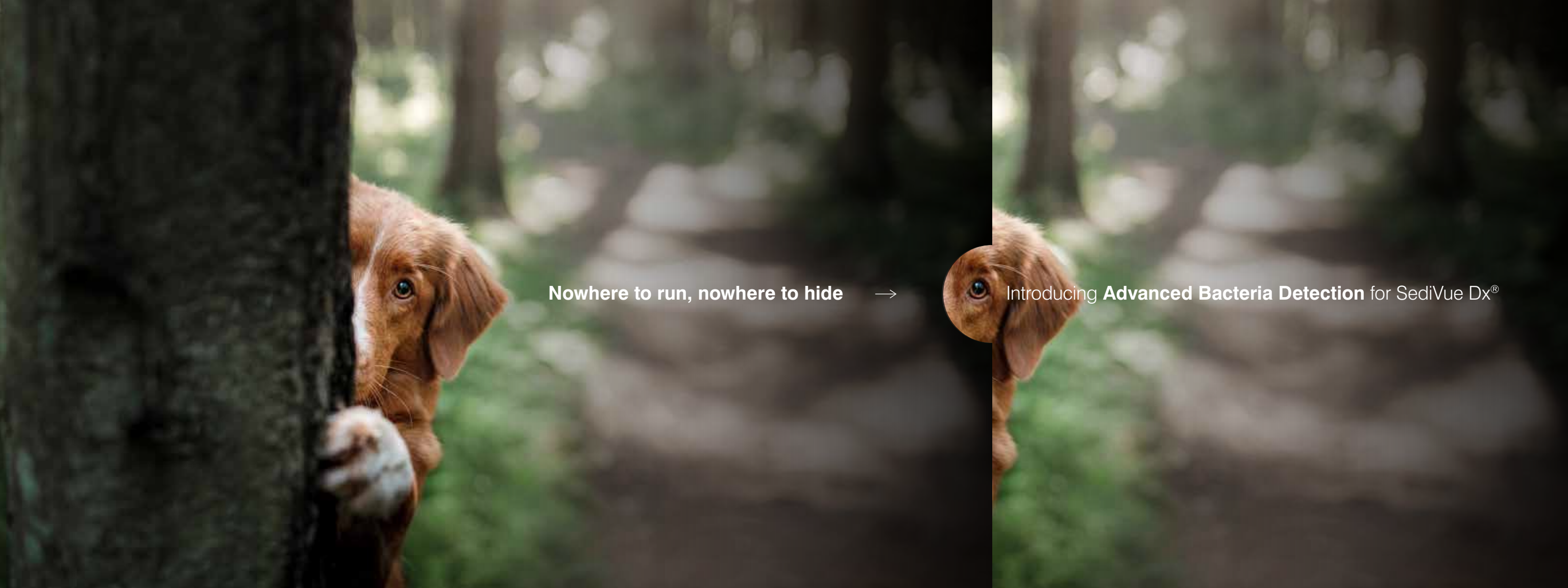

Nowhere to run, nowhere to hide



Introducing Advanced Bacteria Detection for SediVue Dx[®]

A breakthrough innovation for more conclusive and accurate results

Advanced Bacteria Detection with the SediVue Dx[®] Urine Sediment Analyzer: Two advanced, proprietary technologies that work together to seek out bacteria and help you determine next steps quickly. Both yours at no additional cost.

The SediVue[®] Bacteria Confirmation Kit—arriving in your practice soon

Eliminates suspect presence by chemically dissolving blood cells and crystals that interfere with bacteria identification.

Neural Network 5.0 software release—now powered by 350 million images

With twice as many images as the previous release, your SediVue Dx detects bacteria with unprecedented specificity.

← **See the difference**

.....
Learn how and when to use the SediVue Bacteria Confirmation Kit today at idexx.com/svconfirm
.....



Your **SediVue[®] Bacteria Confirmation Kit** will be arriving in the **next 2 weeks**.

Learn more at
idexx.com/svconfirm




One IDEXX Drive
Westbrook, Maine 04092

PRESORTED FIRST-CLASS
US POSTAGE
PAID
IDEXX Laboratories, Inc.

Advanced Bacteria Detection for the SediVue Dx® Urine Sediment Analyzer

Bacteria have nowhere to hide

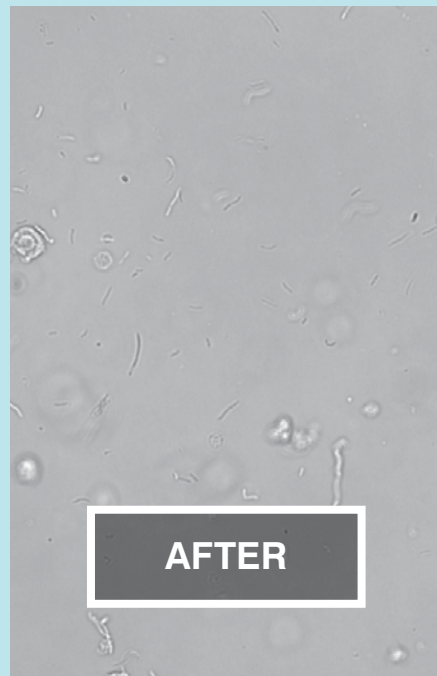

A breakthrough innovation for more conclusive and accurate results

Advanced Bacteria Detection with the SediVue Dx® Urine Sediment Analyzer: Two advanced, proprietary technologies that work together to seek out bacteria and help you determine next steps quickly. Both yours at no additional cost.

The SediVue® Bacteria Confirmation Kit—arriving in your practice soon
Eliminates suspect presence by chemically dissolving blood cells and crystals that interfere with bacteria identification.

Neural Network 5.0 software release—now powered by 350 million images
With twice as many images as the previous release, your SediVue Dx detects bacteria with unprecedented specificity.

← **See the difference**

.....
Learn how and when to use the SediVue Bacteria Confirmation Kit today at idexx.com/svconfirm
.....

Your **SediVue® Bacteria Confirmation Kit** will be arriving in the **next 2 weeks**.

Learn more at idexx.com/svconfirm



One IDEXX Drive
Westbrook, Maine 04092

PRESORTED FIRST-CLASS
US POSTAGE
PAID
IDEXX Laboratories, Inc.

Nowhere to run, nowhere to hide

Introducing **Advance Bacteria Detection** for SediVue Dx.
A breakthrough innovation for more conclusive and accurate results.

[See how it works](#)

Detect bacteria and determine next steps quickly, at no additional cost

We are committed to making sure that your investments appreciate over time. That's why we developed this advanced, proprietary technology and why it's yours at no additional cost.


Introducing the Advanced Bacteria Detection Kit

Simply add one drop of each reagent in the kit and watch debris dissolve. The kit eliminates uncertainty about bacteria results.

Announcing the release of Neural Network 5.0

Powered by 350 million images, twice the size of the last release, your SediVue Dx detects bacteria with unprecedented specificity.

Be ready to use your Bacteria Confirmation Kit when it arrives


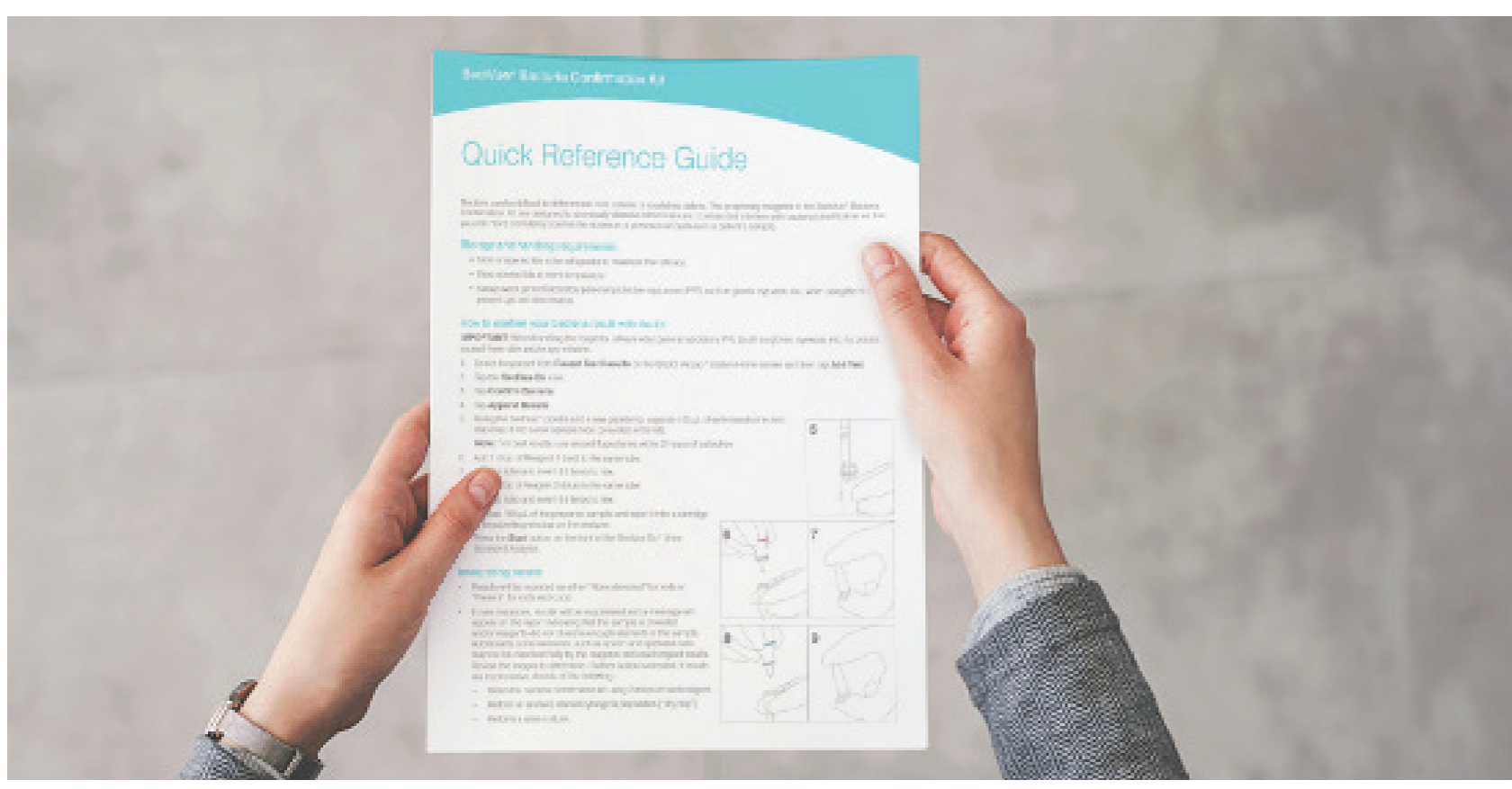


[Download our step-by-step guide](#)



Watch and learn the simple workflow

See how Advanced Bacteria Detection brings clarity to your results

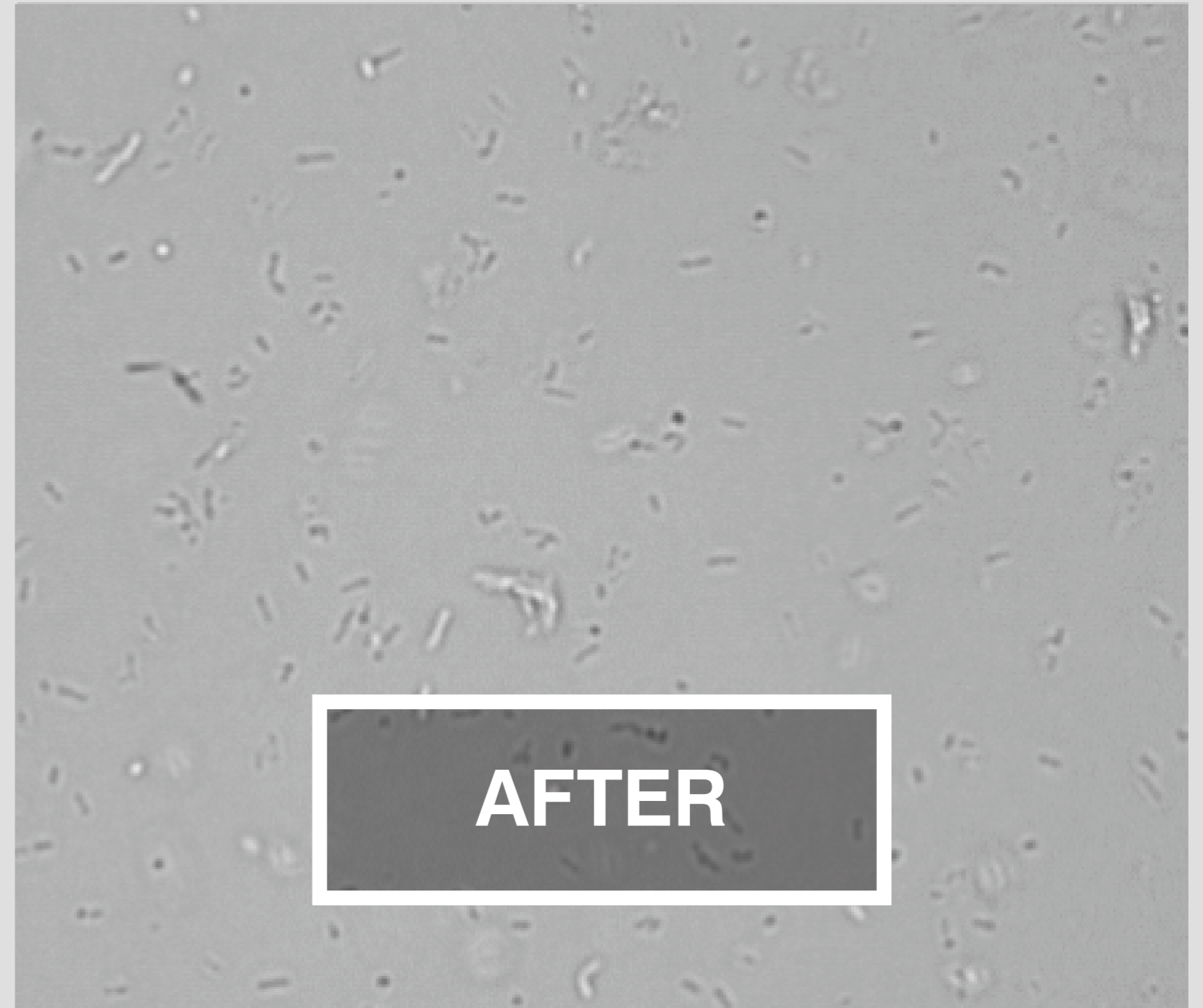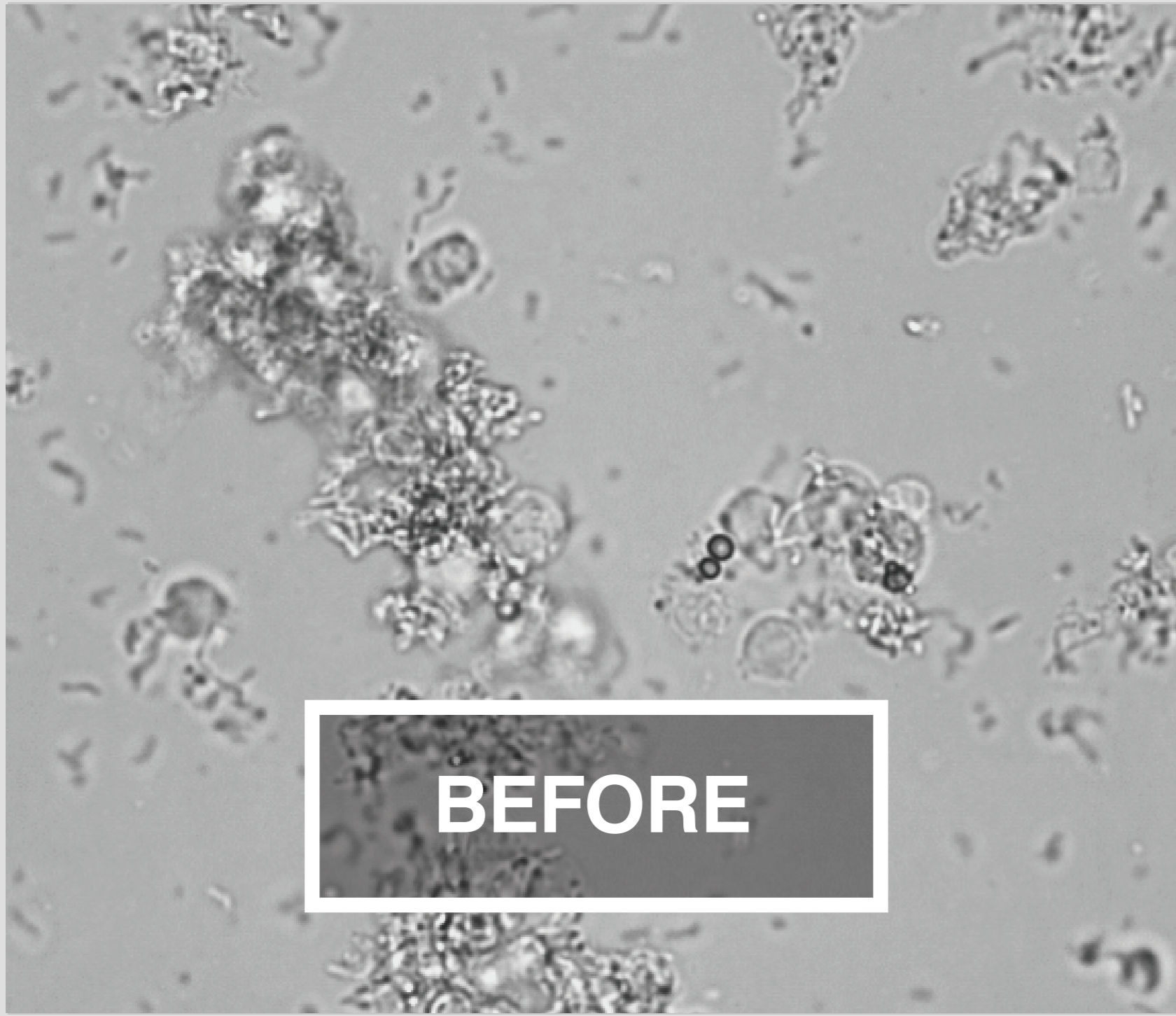

Frequently Asked Questions

Q When should I use the SediVue Bacteria Confirmation Kit?

A Use the kit if a bacteria result:

- Is reported as "Suspect presence" and the corresponding images do not provide conclusive evidence.
- Does not align with the corresponding images and/or your expectations.

Q How does the Bacteria Confirmation Kit help to clarify my bacteria results?

A The reagents in the kit dissolve crystalline and cellular material. This provides greater visibility to the presence or absence of bacteria.

Q How do I interpret the results I get after using the bacteria confirmation kit?

A Results for rods and cocci will be reported as either "None detected" or "Present." As always, be sure to use the corresponding images along with results to make the most informed decision.

Q Will I be charged for running the Bacteria Confirmation Kit?

A You will not be invoiced for runs that use the Bacteria Confirmation Kit if those runs are part of a complete urine sediment analysis.

[See how it works](#)

Would it help to speak with someone?
Call our customer support team at **1-888-79-IDEXX**

This is FPO: Pick up Form from <https://app.hubspot.com/content/413558/edit-beta/4198594946>

Two Column


Last Name

Three Column

Postal Code

IDEXX Account number

[Submit](#)

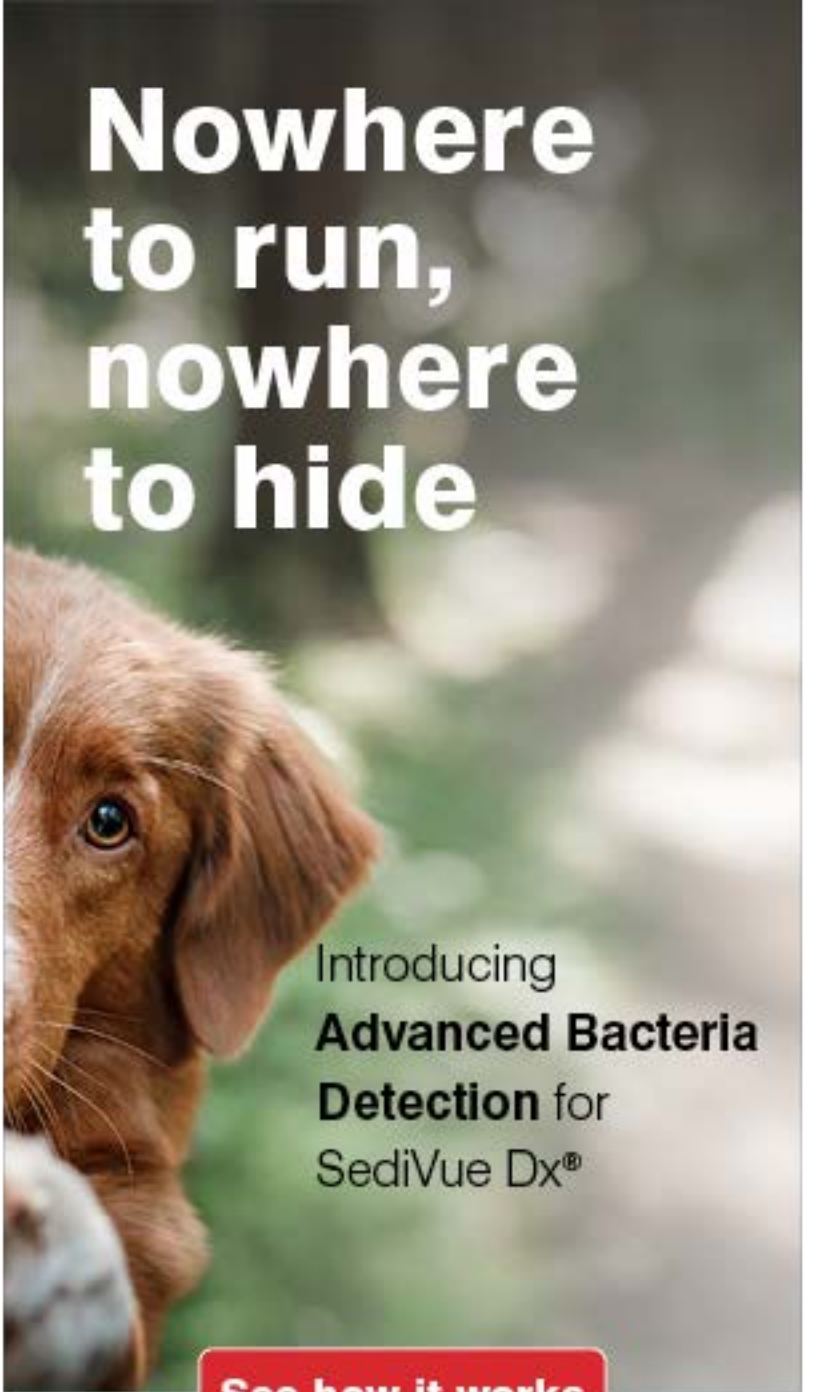

**Nowhere
to run,
nowhere
to hide**

Introducing
**Advanced Bacteria
Detection** for
SediVue Dx[®]

See how it works

IDEXX